



Encore® D/C makes core buildup easier.

Deliver direct to the bottom of the post hole with the exclusive needle tip.

Encore D/C is a dual-cure composite core buildup material in an automix cartridge. The non-slumping formula eliminates the need to use a matrix and the material can be light-activated in order to accelerate the set. Available with fluoride and in natural and contrast shades.

- Automix and dual-cure formula speeds procedure and insures depth of cure
- Light-activate for 20 seconds to accelerate the set
- Needle tip allows delivery of material to bottom of a post hole
- Non-slumping formula so there is no need to use a matrix
- Cuts like dentin to reduce ditching
- Available in two shades – natural and contrast
- Radiopaque

Comprehensive strength:	222.35 Mpa
Flexural strength:	110.00 Mpa
Diametral tensile strength:	48.50 Mpa
Light-activated depth of cure:	2 mm. 40 seconds

Ordering is easy. Call 800-235-5862 or contact your preferred dealer. Visit centrixencore.com for more information.



CENTRIX
Encore® D/C
Dual-Cure Core Buildup Material



centrix®
Making Dentistry Easier.™

**6-STEP GUIDE TO BUILDING A VOID-FREE,
BONDED, COMPOSITE POST & CORE**

1. Enlarge root canal to accept Encore D/C needletip. Bevel the edges to eliminate sharp angles.



2. Etch the post hole, rinse thoroughly and dry.



3. Apply bonding agent according to manufacturers instructions.



4. Insert needletip to the bottom of the post hole and inject material with slow steady pressure.



5. Continue filling by injecting and slowly withdrawing the needletip.



6. If desired, light-activate Encore D/C to accelerate the set. Allow material to fully cure and then cut and shape the core. The result is a solid, securely bonded post and core.

ORDERING INFORMATION:**ENCORE D/C**

REF 310150 **Natural shade, with fluoride, Refill Kit**
contains one 50g S25 cartridge,
10 Access Combo Nozzles

REF 310152 **Contrast blue shade, with fluoride, Refill Kit**
contains one 50g S25 cartridge,
10 Access Combo Nozzles

SUGGESTED ACCESSORIES*

REF 310156 Type S25 automix gun
REF 910628 Automix nozzles, 50
(yellow hub)
REF 290025 144 Encore D/C Needletips
REF 910631 Access Combo Nozzles, 50
(yellow hub w/needletip)

*Fits Encore D/C, plus other brands of core paste cartridges.

Ordering is easy.
Call 1-800-235-5862
or contact your preferred dealer.

centrixdental.com

RISK FREE **Our Guarantee** — If you are not 100% satisfied, simply return the product for a complete refund. It's that easy.



centrix®

Making Dentistry Easier.™