

What's harder than core buildups?

*Figuring out their cost per unit dose.
Encore DC MiniMix is the only
dual-cure sold by dose.*

Single-Dose. Great price. Problems solved.

Our new single-dose Encore D/C MiniMix kit contains 36 cartridges. Each cartridge has their own mixing nozzle and our exclusive needletip. Our product is dual-cure, for fast light-activation. Plus, the smaller size and ease of use will save you valuable chair time and reduce patient stress.

- Light-activate for 40 seconds for complete cure
- Needletip allows delivery of material to bottom of a post hole
- MiniMix technology virtually eliminates waste
- Non-slumping formula so there is no need to use a matrix
- Cuts like dentin to reduce ditching
- Radiopaque

Compressive strength:	222 Mpa
Flexural strength:	110 Mpa
Diametral tensile strength:	48 Mpa
Light-activated depth of cure:	2mm 20 seconds

Ordering is easy. Call 800-235-5862 or contact your preferred dealer.
Visit centrixcore.com for more information.



CENTRIX
Encore[®] D/C
MiniMix[®]
Unit Dose Dual-Cure Core Buildup Material



centrix[®]
Making Dentistry Easier.SM

CENTRIX
Encore[®] D/C
MiniMix[®]

Unit Dose Dual-Cure Core Buildup Material

**6-STEP GUIDE TO BUILDING A VOID-FREE,
 BONDED, COMPOSITE POST & CORE**



1. Enlarge root canal to accept Encore D/C MiniMix. Bevel the edges to eliminate sharp angles.




2. Etch the post hole, rinse thoroughly and dry.



3. Apply bonding agent according to manufacturer's instructions.



4. Insert needletip to the bottom of the post hole and inject material with slow steady pressure.

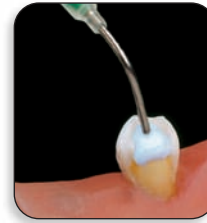

5. Continue filling by injecting and slowly withdrawing the needletip.



6. If desired, light-activate Encore D/C to accelerate the set. Allow material to fully cure and then cut and shape the core. The result is a solid, securely bonded post and core.



ORDERING INFORMATION:

ENCORE D/C MINIMIX

Contains: 36 single-dose cartridges (0.5ml net ea) and 36 mixing nozzles

REF 310254 Natural shade, 36



REF 310255 Contrast blue shade, 36



SUGGESTED ACCESSORIES*

REF 130651 Snub Nose Snap-Fit[™] Syringe



RISK FREE Our Guarantee — If you are not 100% satisfied, simply return the product for a complete refund. It's that easy.

Ordering is easy.
 Call 1-800-235-5862
 or contact your preferred dealer.

centrixdental.com



centrix[®]

Making Dentistry Easier.SM