



How about just staining caries?

With Exposé,[®] there is no measuring or mixing, and no need to stock vinegar for those messy accidents.

Exposé is a caries indicator built into an applicator and dispensing system! The detector solution is pre-applied to our Benda[®] Micro applicators. Apply Exposé simply by touching a wet tooth surface, and Exposé stains infected dentin. No dispensing, no dripping, no mess! Exposé eliminates the dreaded mess of bulk containers or brush-tipped syringes. At Centrix we make dentistry easier.

- No set-up, no cleanup, and no cross contamination
- Supports minimally invasive dentistry
- Comes in two head sizes
- Available in 2 indicator colors: Blue and Red



Exposé is packaged in our Benda[®]Matic pop-up dispenser.

Ordering is easy. Call 800-235-5862 or contact your preferred dealer. Visit centrixdental.com for more information.



CENTRIX
Exposé[®]
Caries Indicator



centrix[®]
Making Dentistry Easier.SM

INSTRUCTIONS FOR USE

- 1) Remove an Exposé applicator from the Benda®Matic.
- 2) Lightly wet the Exposé applicator with either water or the wet tooth.
- 3) Apply the wet Exposé applicator to the prepared tooth.
- 4) Rinse and examine the area for any Exposé that remains on the affected dentin.
- 5) Proceed as usual.



Blue Indicators



Red Indicators



ORDERING INFORMATION:

Exposé with Dispenser

- REF 310322 Blue Indicator 96 X-Fine and 96 Fine, each in their own BendaMatics
- REF 310325 Red Indicator 96 X-Fine and 96 Fine, each in their own BendaMatics

Exposé Refills

- REF 310330 Blue Fine Caries Indicator Refill, 576
- REF 310331 Blue X-Fine Caries Indicator Refill, 576
- REF 310332 Red X-Fine Caries Indicator Refill, 576
- REF 310333 Red Fine Caries Indicator Refill, 576

RISK FREE Our Guarantee — If you are not 100% satisfied, simply return the product for a complete refund. It's that easy.

Ordering is easy.
Call 1-800-235-5862
or contact your preferred dealer.

centrixdental.com

