



## Dip. Mix. Brush. Throw away.

*Centrix makes fluoride varnish  
safer and cleanup easier.*

FluoroDose features a 5% sodium fluoride cavity varnish in the single-use LoliPack® package, the maximum allowed fluoride dosage and is FDA-approved for treating dentinal sensitivity. Each unit contains one single dose (0.3ml) of FluoroDose plus one clean Benda® Brush applicator which allows you to mix varnish prior to application guaranteeing a consistent level of fluoride in every application. Simply peel back the cover, dip in the brush, mix and apply.

The ADA considers fluoride varnish to be safe and efficacious as part of a caries prevention program that includes caries diagnosis, risk assessment, and regular dental care.

- Unit dose delivery allows for ease of use and cleanup
- Single step brush-on procedure is easier and safer to use than gels and rinses
- No need for a prophylaxis before application
- Dries in seconds upon contact with saliva
- Remains on tooth for 6 to 8 hours for optimum fluoride uptake
- Available in great tasting Bubble Gum and Mint flavors
- 5 percent sodium fluoride (NaF) or 22,600 ppm of fluoride
- Best value on the market

Ordering is easy. Call 800-235-5862 or contact your preferred dealer.  
Visit [centrixdental.com](http://centrixdental.com) for more information.

CENTRIX  
**FluoroDose**™  
5% Sodium Fluoride Varnish



**centrix**®  
Making Dentistry Easier.™



CENTRIX  
**FluoroDose™**  
 5% Sodium Fluoride Varnish

**Dip. Mix. Brush. Throw away.**

Centrix makes asepsis and post-procedural cleanup easier. No more messy gels and uncomfortable trays; no more inherent worry of fluoride ingestion.

## APPLICATIONS

### Indications for use:

- Effective in the prevention of tooth decay when used in conjunction with a comprehensive oral hygiene program
- Helps relieve tooth sensitivity when applied to exposed roots
- Prevention of root decay in older patients who experience difficulty in cleaning their teeth

### Contraindications

Children with a low risk of cavity formation who consume optimally fluoridated water or children who receive routine fluoride treatments through a dental office.

The varnish application should be repeated at three-month intervals for high-risk children and at six-month intervals for children who are not at high risk.

There are several studies at the ADA website explaining the uses of fluoride varnishes, visit: [http://www.ada.org/prof/resources/topics/science\\_fluoride\\_varnish.asp](http://www.ada.org/prof/resources/topics/science_fluoride_varnish.asp)

### Directions for use:

1. Clean teeth and remove excess moisture from area to be treated.
2. Grip the foil cover and slowly peel back, exposing the brush applicator and varnish well.
3. Remove applicator brush and stir material in well.
4. Paint a thin film of varnish onto treatment area.
5. Let the varnish dry for approximately 10 seconds and then instruct the patient to close their mouth.
6. Dispose package and applicator after use.
7. Patient may leave immediately after treatment. Patient should be advised to remain on a soft food diet and only drink cold liquids for two hours after treatment; and not to brush for a minimum of 4 to 6 hrs.
8. When dry, any excess varnish may appear "spotty" and may slide off the tooth surface. This will not affect the efficacy of the product as there is still a full layer of varnish on the tooth.

## ORDERING INFORMATION:

REF 360077      **FluoroDose Bubble Gum, 600**  
 REF 360087      **FluoroDose Bubble Gum, 120**

REF 360078      **FluoroDose Mint, 600**  
 REF 360086      **FluoroDose Mint, 120**

Product covered by one or more of the following patents: US 5,001,803; US 5,660,273; US 6,116,414; US 6,328,159; US 6,685,013; US 6,959,808; US 7,243,789, US D421,217; Japan 3877764 ,EP 077021; EP 1035033; and EP 1197442. Other US and International patents pending or granted.



**Our Guarantee** — If you are not 100% satisfied, simply return the product for a complete refund. It's that easy.



### Ordering is easy.

Call 1-800-235-5862  
 or contact your preferred dealer.

[centrixdental.com](http://centrixdental.com)



**centrix®**

Making Dentistry Easier.™