

## RETRACTION TECHNIQUE FOR SINGLE PREPS



### STEP 1

Prepare the tooth in your standard procedure.



### STEP 2

If there is excessive bleeding or oozing use a Centrix Benda® Micro applicator to burnish in an astringent/hemostatic agent.



### STEP 3

Choose the correct size of GingiCap. Make sure that the GingiCap fits around and covers the whole tooth.



### STEP 4

Inject the GingiTrac™ retraction paste around and over the prepared tooth.



### STEP 5

Fill the GingiCap with GingiTrac retraction paste and place the filled GingiCap over the tooth, making sure that it goes on all the way. Have the patient bite and hold for 3-5 minutes depending on the amount of bleeding/oozing.



### STEP 6

Remove the GingiCap and check the retraction. If it is satisfactory, take your standard impression. If more retraction is desired, repeat steps 3 to 6.



### STEP 7

Ready for impression.



Available in large-size and regular-size.

CENTRIX  
**GingiCap™**



**centrix®**

*Making Dentistry Easier.™*

Ordering is easy: call 800-235-5862, or contact your preferred dealer.  
Log on to [gingitrac.com](http://gingitrac.com) for more information.

## RETRACTION TECHNIQUE FOR MULTIPLE PREPS




### STEP 1

Prepare the tooth in your standard procedure.



### STEP 2

If there is excessive bleeding or oozing use a Centrix Benda® Micro applicator to burnish in an astringent/hemostatic agent.

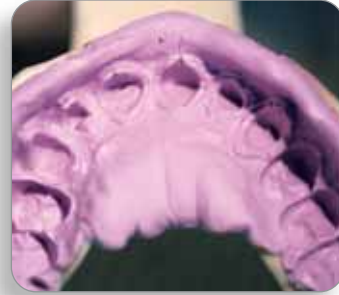

### STEP 3

Express GingiTrac Matrix into a plastic tray and place over the prepared teeth. Use a cotton roll on the opposite side to prevent full closure. Matrix will set in 30 seconds. Remove from the mouth, rinse and dry.



### STEP 4

Inject the GingiTrac™ retraction paste around and over the prepared teeth.



### STEP 5

Fill GingiTrac Matrix with GingiTrac Retraction material and place back over the preparation. Have patient close with firm but not excessive pressure. Hold for 3 to 5 minutes, depending on the amount of bleeding or oozing for complete set and retraction.



### STEP 6

Remove the set GingiTrac. Check for complete retraction. If more retraction is desired, repeat step 5 for 3-5 minutes.

#### HELPFUL HINTS:

- Leave mixing nozzles on cartridges until next use.
- Cartridges require DS50 4:1/10:1 mixing gun.
- Use vinyl gloves, do not handle either material with latex or rubber gloves.

Ordering is easy: call 800-235-5862, or contact your preferred dealer.  
Log on to [gingitrac.com](http://gingitrac.com) for more information.



**centrix**®  
Making Dentistry Easier.™