

Cleans, conditions and lubricates instruments.



*Centrix makes
sculpting composites easier.*



Introducing the LubeCube.™ Clean and lubricate your sculpting instrument by simply running it through the slit on top.

Put an end to composite stickiness, pull-back and unnecessary voids. With LubeCube, simply peel off the adhesive back, stick it to a surface and slide your instrument tip through the slit. The cleaning and lubricating action will allow you to shape with ease. At Centrix, we make dentistry easier.

- Cleans, conditions and lubricates instruments with an unfilled, light-activated, Bis-GMA/UDMA resin monomer
- Lubricant is compatible with all light-activated and dual-cure composites. No alcohol, no solvents.
- Self-adhesive back allows placement anywhere that is convenient
- No-drip dispensing of lubricant micro-coats each instrument
- Sculpt composites faster and easier, with less "pull-back"
- Makes any composite instrument "non-stick"
- Opaque foam cube keeps resin lubricant from polymerizing
- Low cost, single use and disposable
- Smoother, less porous composite surfaces (see illustration on reverse side)

Ordering is easy. Call 800-235-5862 or contact your preferred dealer.
Visit centrixdental.com for more information.




CENTRIX
LubeCube™
Composite Instrument Cleaner and Lubricator




centrix®
Making Dentistry Easier.™

SMOOTHER, LESS POROUS SURFACES!

With LubeCube

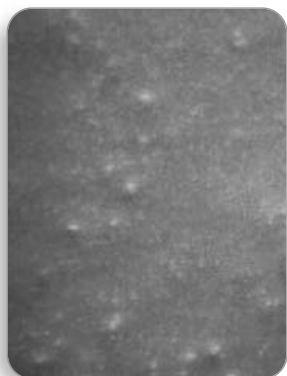

Without LubeCube

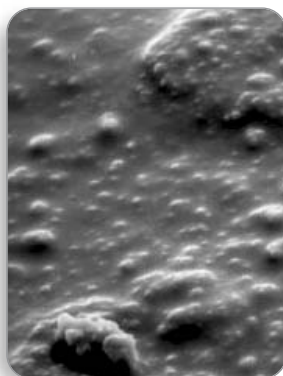


Light-activated composite 750x magnification

The LubeCube cleans and lubricates your sculpting instrument to make shaping composites faster and easier.


LUBECUBE INSTRUCTIONS:




1. Peel backing off bottom of LubeCube.

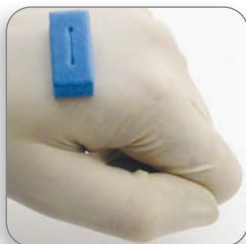

2. Place and/or stick LubeCube in desired location (back of hand, bib, tray, etc).

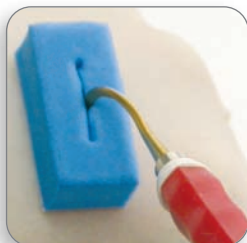

3. Insert tip of composite instrument into slit of LubeCube.
 Glide tip of composite instrument back and forth in slit to clean and lubricate.

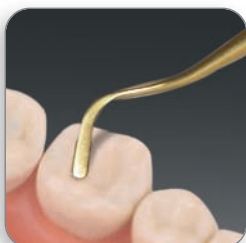

4. Remove composite instrument and begin sculpting the restoration.
Repeat steps 1 thru 4 as desired until restoration is complete.
Dispose of used LubeCube.

ORDERING INFORMATION:

LUBECUBE
 REF 310108 72 LubeCubes



RISK FREE Our Guarantee — If you are not 100% satisfied, simply return the product for a complete refund. It's that easy.

Ordering is easy.
 Call 1-800-235-5862
 or contact your preferred dealer.

centrixdental.com



centrix®
 Making Dentistry Easier.™