

# Direct dispensing. Single component. Moisture activated.

*Centrix makes the mixing, waste and inconsistency of temporary cement a thing of the past. Guaranteed.*



NoMIX® Temporary Cement was designed with both the dentist and patient in mind.

NoMIX's direct syringe delivery allows for exact placement of cement and eliminates the mixing and messy clean up of traditional temporary cements. Patients will appreciate the convenient, .50gr single-dose, take-home packet that's so easy to use it can be applied at home in case of an emergency. NoMIX may be used with crowns made of temporary or permanent materials (acrylic, composite, PFM, etc).

The new unit dose take-home packet can also be directly dispensed for a single provisional restoration in or out of the office.

- Single-component, moisture-activated, non-eugenol, temporary white cement
- Expands slightly upon setting
- Low film thickness
- No mixing, no mess
- Eliminates waste
- Easy to remove from prep



Ordering is easy. Call 800-235-5862 or contact your preferred dealer. Visit [centrixnomix.com](http://centrixnomix.com) for more information.



CENTRIX  
**NoMIX**®  
Temporary Cement  
Moisture-Activated Temporary Cement  
for Crowns & Bridges



**centrix**®  
Making Dentistry Easier.™

CENTRIX  
**NoMIX**<sup>®</sup>  
 Temporary Cement

Moisture-Activated Temporary Cement  
 for Crowns & Bridges

**Benefits:**

- **Moisture-activated formula** — eliminates dry cementing and reduces post-operative sensitivity
- **Prefilled 1ml syringe** — perfect for in-office use without the mixing and clean up
- **Unit dose, take-home packet** — simple to use, easy-to-tear package; easy to carry, same size as credit card, fits in wallet
- **No eugenol** — won't inhibit resins from setting
- **Working time 5 minutes** — final set 15 minutes

**Active Ingredients:**

- **Potassium Sulphate** — desensitizing effect on the tooth
- **Zinc Oxide** — Anti-bacterial ingredient, prevents the “black fungus” and corresponding bad breath
- **Calcium Sulphate** — expands slightly for minimal leakage

The convenient take-home packet provides your patient with an “insurance card” to reduce emergency visits and offers an immediate solution when traveling.

**NoMIX Syringe Technique:**

- Leave prepared tooth and the inside of crown moist, not wet.
- Lightly cover the inside of the crown and the margins of the crown with NoMIX Temporary Cement.
- Set the crown and check for fit. Have patient bite down.
- Wait 5 minutes; cement will be set enough to clean off the excess. Clean the interproximals by knotting floss and pulling through (NOT up).
- The cement will fully set in approximately 15 minutes. It is best to instruct the patient to not eat for at least 1 hour.
- Crown needs to have a good fit or good seal at gingiva. NoMIX should hold with most materials. May be used with crowns made of temporary or permanent materials (acrylic, composite, PFM, etc). Not suitable for pre-formed shell crowns (polycarbonate or metal).

**ORDERING INFORMATION:**

- REF 310520 **NOMIX STARTER KIT**  
 6 x 1ml prefilled syringes plus  
 24 x .50gr unit dose “take-home” cards
- REF 290040 **NOMIX REFILL KIT**  
 12 x 1ml prefilled syringes
- REF 310157 **NOMIX DISPENSER**  
 50 x .50gr unit dose “take-home” cards

**SUGGESTED ACCESSORY**

- REF 640002 NoMIX syringe tips, 30



**Ordering is easy.**

Call 1-800-235-5862  
 or contact your preferred dealer.

[centrixdental.com](http://centrixdental.com)

**RISK FREE** Our Guarantee — If you are not 100% satisfied, simply return the product for a complete refund. It's that easy.



**centrix**<sup>®</sup>

Making Dentistry Easier.<sup>SM</sup>