



RapidCore makes depth of cure fast and worry-free.

Complete depth of cure, without light activation in only two minutes.

It's easy. Simply inject RapidCore through the exclusive mixing nozzle needletip and you are ready to cut the prep in two minutes. Achieve high bond strengths when used with a compatible bonding agent such as Adhere.™ RapidCore's advanced thixotropic formula flows smoothly through the needle yet will not slump, making core forms and matrix bands unnecessary.

- Two-minute set time reduces chair time by 50%
- Pure self cure — no depth of cure concerns
- Needle tip allows delivery of material to bottom of a post hole
- Non-slumping formula so there is no need to use a matrix
- Cuts like dentin to reduce ditching
- Available in three shades – natural, contrast and white
- Radiopaque
- With Fluoride

Bond strength:	21 Mpa with Adhere + Activator
Working time (Room temp.):	60 seconds
Set time in mouth:	2 minutes
Comprehensive strength:	36,000 psi
Water Extractable:	Less than 0.02 mg/cm ² 7 days @ 37°C

Ordering is easy. Call 800-235-5862 or contact your preferred dealer. Visit centrixdental.com for more information.



CENTRIX
RapidCore™

Self-Cure Composite Resin
Core Build-up Material



centrix®
Making Dentistry Easier.™

**6-STEP GUIDE TO BUILDING A VOID-FREE,
BONDED, COMPOSITE POST & CORE**

1. Enlarge root canal to accept RapidCore needletip. Bevel the edges to eliminate sharp angles.



2. Etch the post hole, rinse thoroughly and dry.



3. Apply bonding agent according to manufacturer's instructions.



4. Insert needletip to the bottom of the post hole and inject material with slow steady pressure.



5. Continue filling by injecting and slowly withdrawing the needletip.



6. Allow material to fully cure and then cut and shape the core. The result is a solid, securely bonded post and core.

ORDERING INFORMATION:**RapidCore 50ml Kits include:**

1 50ml cartridge, 15 mixing nozzles
and 15 dispensing tips
REF 310133 White shade
REF 310134 Natural shade
REF 310135 Contrast shade

RapidCore 25ml Kits include:

1 25ml cartridge, 10 mixing nozzles
and 10 dispensing tips
REF 310136 White shade
REF 310137 Natural shade
REF 310138 Contrast shade

SUGGESTED ACCESSORIES

REF 310251 Adhere Adhesive
REF 310252 Adhere Activator
REF 310156 DS25 1:1/2:1 Automix Syringe
REF 360036 DS50 1:1/2:1 Automix Syringe
REF 290025 Automix Tips, 18ga, Green, 144
REF 910628 Mixing Nozzle, Yellow Hub, 50
REF 910631 Combo, Mixing Nozzles, Yellow Hub w/Tip, 50



White shade
Natural shade
Contrast shade



Ordering is easy.
Call 1-800-235-5862
or contact your preferred dealer.

centrixdental.com



Our Guarantee — If you are not 100% satisfied, simply return the product for a complete refund. It's that easy.

Product covered by one or more of the following U.S. patents: 5,052,927; 5,336,088; and D359,119. Other U.S. and International patents pending or granted.

© 2007 Centrix, Inc. All rights reserved. RapidCore and Adhere are trademarks of Centrix, Inc.

MK0037 REV B



centrix®

Making Dentistry Easier.™