ENGLISH

INSTRUCTIONS FOR USE

GingiTrac™ Retraction System

REF 360100 GingiTrac™ Refill Kit
REF 360101 GingiTrac™ Standard
REF 360102 GingiTrac™ Plus
REF 360103 GingiCap™ Refill Regular-Size
REF 360104 GingiCap™ Refill Large-Size

INSTRUCTIONS FOR GINGIVAL RETRACTION

1. Prepare the tooth as normal.
2. If excessive bleeding, use a hemostatic/astringent agent.
3. **For Single Tooth Preparation:** Select appropriate size GingiCap, then proceed to Step 5.
4. **For Multiple Tooth Preparation:**
Purge the Access® Blue (VPS fast-bite impression material) cartridge without the mixing nozzle to ensure that both the catalyst and base are being equally expressed, and then attach the mixing nozzle. Express Access Blue into a plastic tray and place over the prepared tooth. Use a cotton roll on the opposite side to prevent full closure. Access Blue will set in 30 seconds. Remove from the mouth, rinse and dry.
5. Attach the mixing nozzle to the GingiTrac Retraction cartridge, express a "pea-sized" amount and discard. This is to ensure that the catalyst and base are being mixed thoroughly.
6. Fill the GingiCap, or Access Blue impression, with GingiTrac Retraction material. Do not allow filled GingiCap to remain at room temperature for more than 1 minute 30 seconds, otherwise material will harden.
7. Syringe the GingiTrac Retraction material directly around the tooth preparations' margin using the included dispensing tip.
8. Place the filled GingiCap over the preparation.
9. Have patient close with firm but not excessive pressure. Wait 2 minutes, depending on the amount of bleeding or oozing, for complete set and retraction.
10. Remove the set GingiTrac. Check for complete retraction. If needed, repeat steps 6-9.
11. If necessary, gently rinse and dry the prepared teeth. You are now ready to take your standard impression.

- Do not use the GingiTrac™ as a final impression.
- Leave mixing nozzles on cartridges until next use.
- Cartridges require DS50 1:1 mixing gun.
- Do not handle either material with latex or rubber gloves.

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