



NoCord™ VPS

One-Step, Self-Retracting Impressioning System

NoCord™ VPS is a unique impressioning system that makes impression taking easier: Detailed margins and a perfectly-fitted restoration every time!


NoCord™ Wash is the first wash material to incorporate a hemostatic agent to control bleeding and fluid flow.

The unique **NoCord™ MegaBody™** Tray Material is engineered to better drive the Wash and to displace gingiva and completely capture prep margins with the NoCord Wash Material.

**For best results, always use NoCord Wash Material
with NoCord MegaBody Tray Material.**

Visit www.CentrixNoCordVPS.com

centrix®
Making Dentistry Easier.™

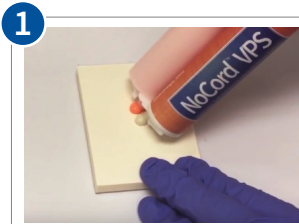

1 Bleed each cartridge prior to attaching SuperMixer™ nozzle to insure proper mix of both materials.



2 Rinse and dry prepped teeth.



3 Fill impression tray with NoCord MegaBody™ Tray Material. A walled dual arch or full arch tray is recommended.



4 Syringe NoCord™ Wash Material into sulcus, generously cover prep and adjacent teeth.



5 **We recommend** use of 'time check beads' to verify full set. Express a small bead of Wash and Tray material.

Start timer when you first express either material.



6 Seat Mega Body Tray Material over the unset NoCord VPS Wash Material. Have patient bite firmly or hold in place with light pressure until material is set.



7 If using 'time check beads', verify full set.

Remove from mouth at 4:45 minutes.



8 Check impression. Fabricate final restoration.

Easy, accurate impressions!

*In extreme bleeding cases pretreat with Access® Edge or AccessFLO® prior to using wash.

*Total set time is based on 98.6°F/37°C patient temp.

*Allow min 2 hours to degas after final set to pour up.

Photos 2,3,4,6 & 8 courtesy of Dr. Jon A. Heezen DDS, Tracy, MN

Ref 310280 NoCord™ VPS Introductory Kit
Ref 310282 NoCord™ Wash Material, 2-Pack
Ref 310284 NoCord™ Wash Material, 10-Pack

Ref 310281 NoCord™ MegaBody™ Tray Material, 2-Pack
Ref 310283 NoCord™ MegaBody™ Tray Material, 10-Pack
Ref 310285 NoCord™ VPS System Refill Kit

For additional information, clinical video, tips and doctors testimonials, visit www.CentrixNoCordVPS.com