

Tempit®

A Family of Temporary
Filling & Sealing Materials

FOUR PROVEN TEMPORARY FILLING MATERIAL FORMULAS, PROVIDING OPTIMAL RESULTS IN ANY CLINICAL SITUATION.



Just as permanent restorations demand specific materials for different needs, temporary restorations also have different requirements, demanding the right material for each case.

Select the right Tempit formula for each case and inject into the prep. Our unique delivery system requires no mixing, spatulation or messy placement. Each Tempit product is packaged in pre-filled unit-dose tips so it can be delivered precisely and perfectly every time, directly into the prep! Tempit saves time, reduces waste and provides the right filling material for each patient's need and every temporary application.


FOUR UNIQUE TEMPORARY FILLING MATERIALS, ONE DELIVERY SYSTEM

- **Tempit** – For Endo access cavity sealing. Moisture activated, expands slightly to completely seal any opening.
- **Tempit-E** – Tempit with Eugenol. Provides relief from pain and discomfort in primary and permanent teeth.
- **Tempit L/C** – Inlay/onlay indirect restorations. Sets firm but flexible so it can be plucked off the restoration with an instrument; no bur required.
- **Tempit Ultra-F** – Long term restorations. Harder wearing for long-term cases and bruxers. Also, ideal as a tooth-colored restorative material for deciduous teeth (prior to exfoliation).

- No mixing, no mess, no cross-contamination
- Unit-dose tips for easy, precise placement and maximum asepsis
- Works with any Centrix C-R® or similar composite gun



TEMPIT IS EASY-TO-USE, DIRECT-TO-THE-PREP

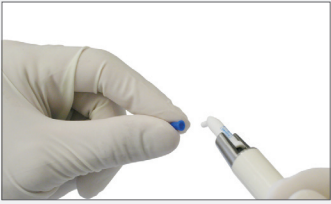

1 Remove cap.



2 Place tip into dispensing gun.



3 Inject directly into prep. Tempit and Tempit-E are moisture activated and both set in 5 minutes. Tempit L/C and Ultra-F are light activated.



4 No cleanup needed. Discard after use.

APPLICATIONS AND FEATURES



Short-Term Temps

Tempit

Moisture-Activated Temporary Filling & Sealing Material

- Ideal for endo access openings and short-term temporaries.
- Moisture-activated, sets in 5 minutes or less.
- Expands slightly when set, seals out microleakage and bacterial ingress.
- Contains calcium sulfate and zinc oxide.



Short-Term Temps with Eugenol

Tempit-E

Moisture-Activated Temporary Filling & Sealing Material with Eugenol

- Palliative eugenol formulation for temporary filling use, especially in cases of reversible pulpal inflammation.
- Anti-inflammatory and local anesthetic effects helps facilitate pulpal healing.
- Moisture activated, sets in five minutes.
- Expands slightly when set to seal out microleakage and bacterial ingress.



Indirect Temp Restorations

Tempit L/C

Light-Activated Temporary Material for Inlays & Onlays

- Perfect for indirect restorations, inlays/onlays.
- Seals implant screw access openings.
- Sets firm yet flexible.
- Easily removed with an explorer without damaging the margins.
- Light-activated, sets in 40 seconds or less.
- No etching or bonding agent required.



Longer-Term Temps

Tempit Ultra-F

Light-Activated, Longer-Term Temporary Material

- Perfect for longer term temporaries.
- Rigid, harder wearing material, with fluoride release.
- High compressive strength for bruxism patients.
- Ideal for tooth-colored restorations in deciduous teeth.
- Less soluble than reinforced ZOE intermediate restorative material.
- Light-activated, sets in 40 seconds or less.
- No etching or bonding agent required.

ORDERING INFORMATION

REF 310060 Tempit, 30 x 0.35g Prefilled Tips

REF 310031 Tempit-E, 30 x 0.35g Prefilled Tips

REF 310063 Tempit L/C, 30 x 0.25g Prefilled Tips

REF 310064 Tempit Ultra-F, 30 x 0.20g Prefilled Tips

REF 310065 Tempit Ultra, 30 x 0.20g Prefilled Tips

REF 310067 Tempit Essentials Intro Kit: 5 x .35g prefilled tips of Tempit, 5 x .35g prefilled tips of Tempit-E, 5 x .25g prefilled tips of Tempit- L/C, and 5 x .20g prefilled tips of Tempit Ultra-F

Ordering is easy. Call **800.235.5862** or contact your preferred dealer.

Visit **centrixdental.com** for more information.